


Sign up for a DTX Studio™ Clinic Starter License




1

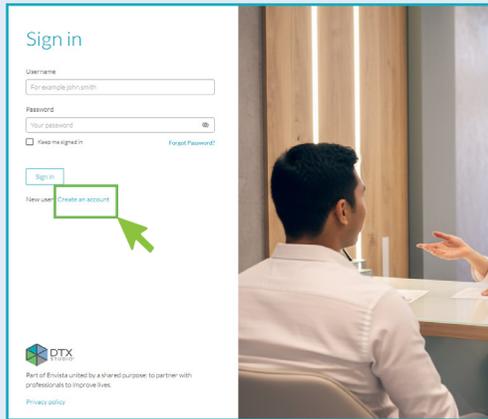


Go to go.dtxstudio.com. If you have no DTX Studio account yet, click on **Create an account**.

2

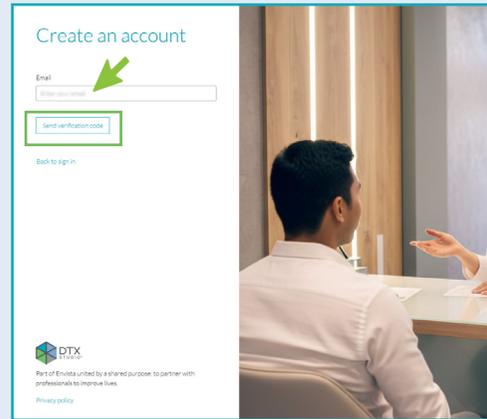


Enter your **Email** address and click **Send verification code**.

3

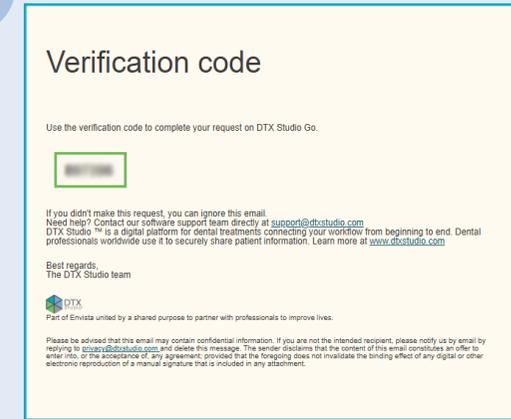


You will receive an email (no-reply@dtxstudio.com) with the verification code. Copy this code.

4

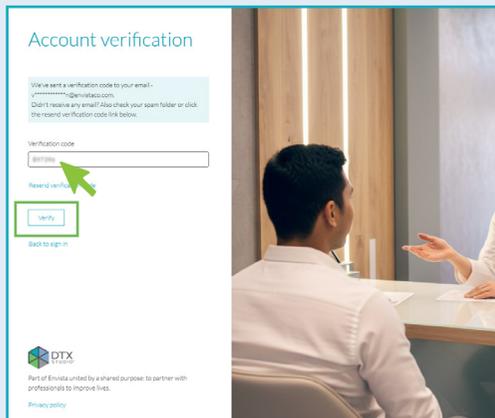


Paste it into the **Verification code** field and click **Verify**.

5

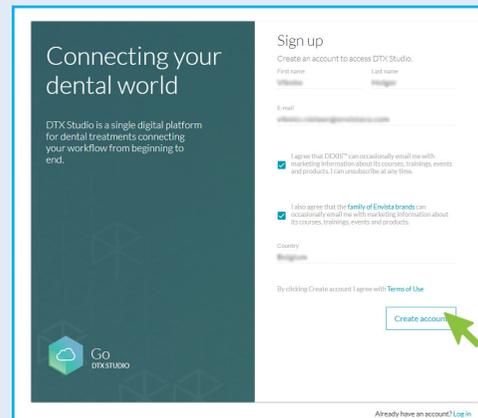

Fill in the requested details and click **Create account** to start creating a DTX Studio Go account.

6

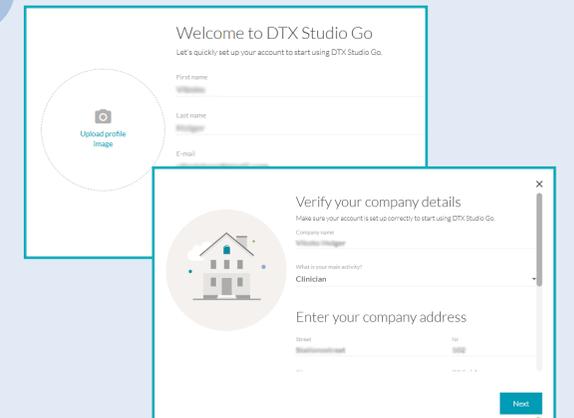


Follow the steps to customize your DTX Studio Go account. Click **Next**.

Sign up for a DTX Studio™ Clinic Starter License




7

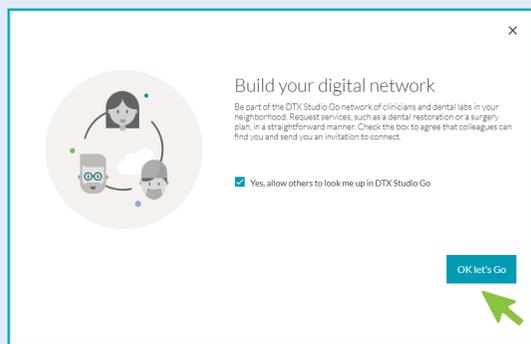


Enter your company details and click **Next**. Then click **Ok let's Go**.

8

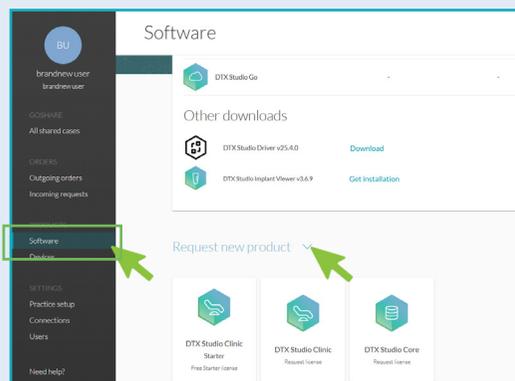


1. In the left sidebar, click **Software**.
2. Click on the drop-down arrow next to **Request new product** if no product tiles are shown.

9

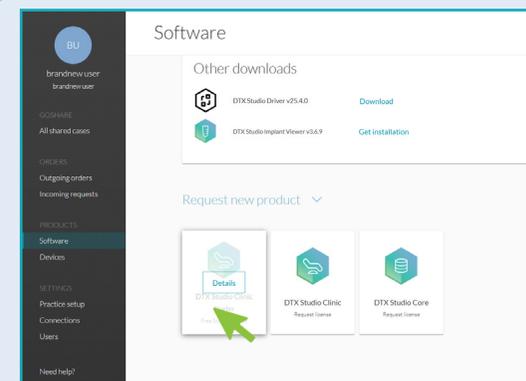


Click on **DTX Studio Clinic Starter Free Starter License**.

10

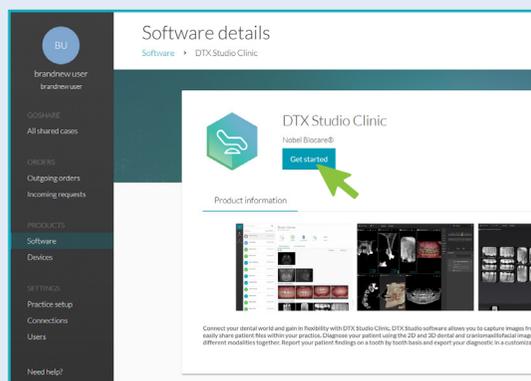


Click **Get Started**. This will take you to the DTX Studio Clinic Installation details.

11

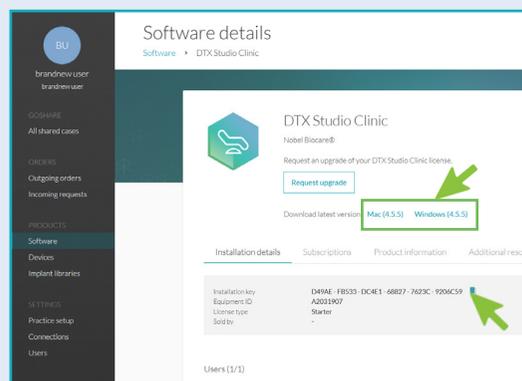

Click **Mac** or **Windows** to download the proper latest version. Copy the installation key from this page. You will need it during installation.

12

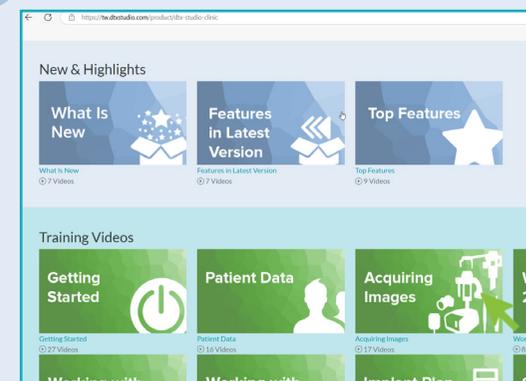

If you need further help with the installation, you can turn to the DTX Studio Clinic **Getting started** section on our training website tw.dtxstudio.com.