


How to Subscribe to the 2D Diagnostics Module




1

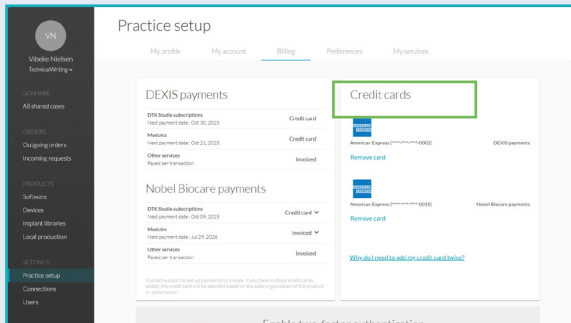


1. Log in to DTX Studio™ Go (go.dtxstudio.com).
2. Go to **Practice Setup** > **Billing**.
3. Ensure a credit card is added to your practice.

2

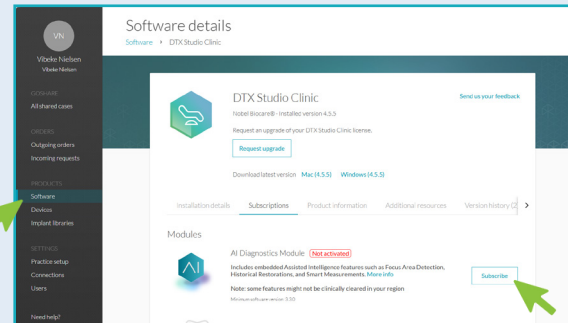


1. Go to **Software** > **Subscriptions**.
2. Click **Subscribe**.
3. Confirm.

3

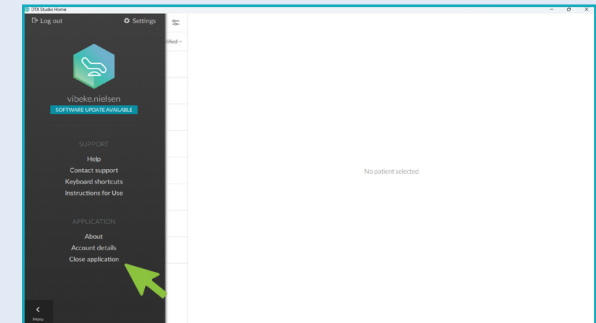


Close DTX Studio™ Clinic via **Menu** > **Close application** and start DTX Studio Clinic again for the subscription to take effect.

4

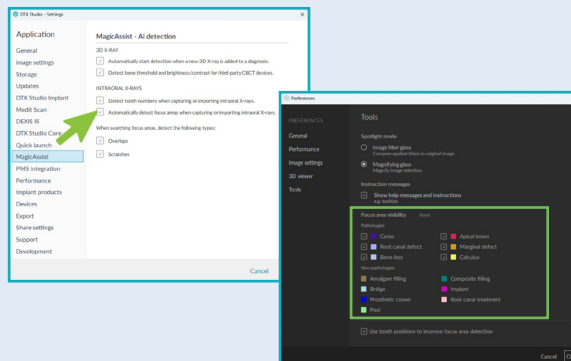


In DTX Studio™ Home and in DTX Studio Clinic, choose your **MagicAssist™ AI detection** settings and **Tools** preferences.

5

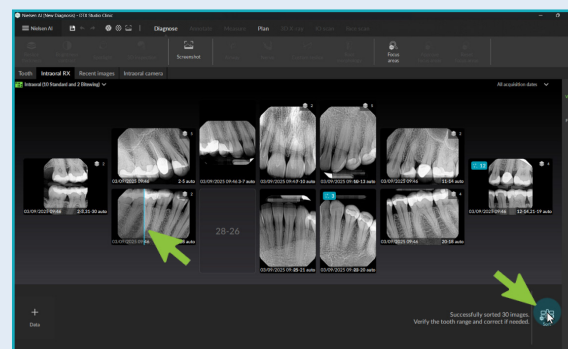

2D AI dental findings detection starts automatically for newly acquired, imported, or migrated images. It can be manually triggered for existing images.

6

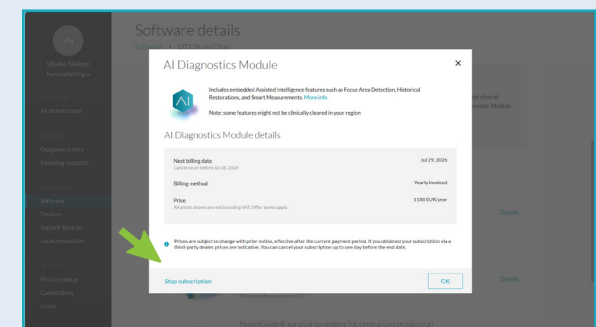

To stop the subscription:

1. Go to **Software** > **Subscriptions** on DTX Studio Go.
2. Click **Details** > **Stop subscription**.